


Pipette Guide for **MICROSAFE®** Tube

1. Prick your finger


2. Grab the tube from the end

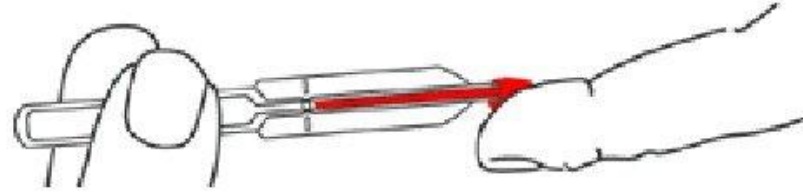

a. Bring tube horizontally to the sample (Do not squeeze the end)

b. The tube will automatically suck up the blood
(all the way to the black lines)

Tip: That should be enough blood for meter to accept the sample.

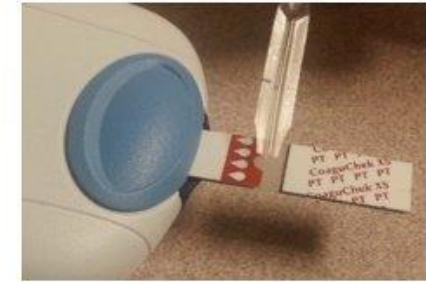

3. With meter sitting on a flat surface align tube to the strip.

a. Make sure the tip is aiming on the clear plastic area of the strip.

b. While aiming in the clear area, turn tube vertically and let the blood flow in the clear area.

Tip: If blood is not flowing automatically into the area, hold the pipette with one hand, and with your other hand, slowly squeeze the tube starting from the end all the way to the middle section



4. **DO NOT** remove tube right away — Wait until you hear a *beep* or if there is an hourglass  on the screen.

